

FACE-SHIELD/LUNG BAG INSTALLATION

1. Unlatch the torso of the Prestan Professional Manikin by popping the 'tabs' located at the top of the shoulders (Figure 1). Carefully lift the torso frame open by the neck.
Note: Do not pull on tabs (past popping the latch). Doing so may break the tabs or stretch out the frame over time, impairing the latching function.



Figure 1



Figure 2

2. Attach the head to the torso by placing the neck of the head onto the two spindles on the neck portion of the torso (Figure 2).
3. Fold the Lung-Bag portion of the Face-Shield/Lung-Bag (Figures 3a-3d):
 - The Face-Shield/Lung-Bag has four 'dotted lines' printed on one side of the bag (along with an oval)
 - Using the four dotted lines, fold the Face-Shield/Lung-Bag into fourths, following the diagram below

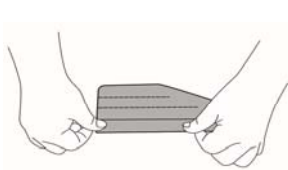


Figure 3a

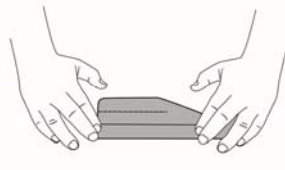


Figure 3b

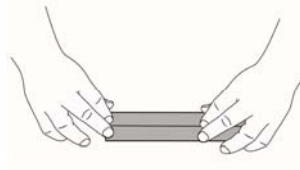


Figure 3c

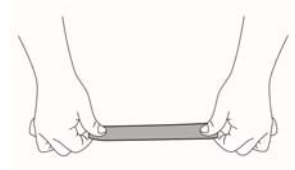


Figure 3d

4. Tilt back the head of the manikin (Figure 4).
5. Insert the folded Lung-Bag into the mouth of the manikin (Figure 5).
6. Push the Lung-Bag into the mouth until the bag exits through the neck of the manikin (Figure 6).
7. Grasp the Lung-Bag and finish pulling it through the neck of the manikin (Figure 7).
8. Attach the Face-Shield tabs to the 'buttons' located on the side of the manikin head (Figure 8).
9. Lightly pull down on the Lung-Bag to help open the mouth hole on the Face-Shield.
10. Then position the Lung-Bag on top of the compression plate (the hard plastic circular piece found on top of the torso foam core) with the circular imprint on the Lung-Bag aligning with the compression plate (Figure 9).
11. With proper positioning, the neck of the Face-Shield/Lung-Bag will have slack. Make sure it is not twisted.
12. Close the torso and secure the latch by pushing on the shoulder tabs until they snap into place.

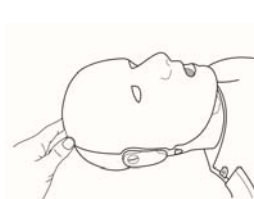


Figure 4

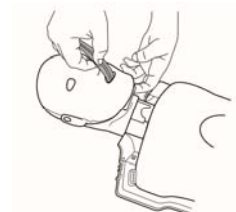


Figure 5

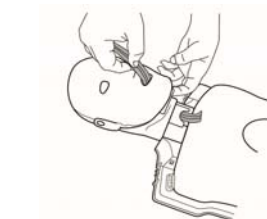


Figure 6

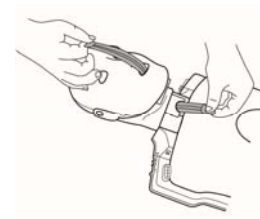


Figure 7



Figure 8

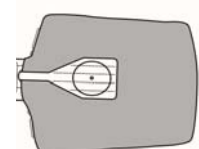


Figure 9