

Prestan Professional Manikins

Replacing the Prestan Infant Clicker

To replace the Clicker on the Infant Manikin, you will need a small Flat Head Screwdriver.

The clicker is the round disk in the middle of the manikin's back. To get to the clicker, first peel down the back skin (*Figure 1*).

Remove the black padded covering to expose the clicker (*Figure 2*).

Disconnect the wiring below the clicker by squeezing the black connector and pulling upward and out (*Figure 3*).

Using a flat head screwdriver, gently pry the clicker from the infant by inserting the flat edge into one or more of the three slots. The clicker will pop out (*Figure 4*).

Push the new clicker into place. Plug the wiring back in, pushing the connector until a click is heard.

Replace the black foam pad and roll the skin back into place from the bottom moving upward, making sure to attach and firmly push the skin into place at all the contact points.

Note that the infant skin is fairly loose around the manikin at the sides and shoulders.


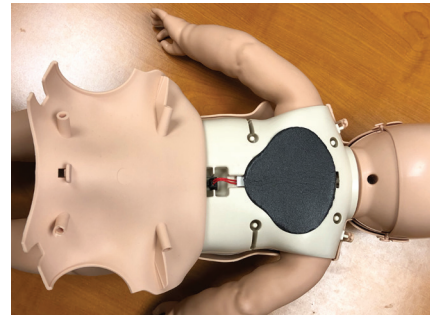


Figure 1

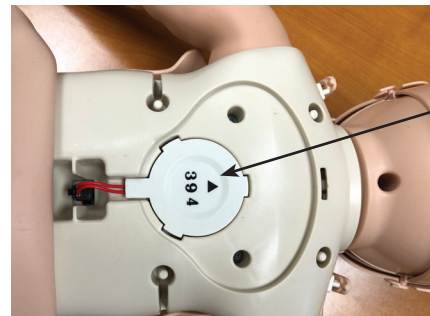

Clicker
in Manikin

Figure 2


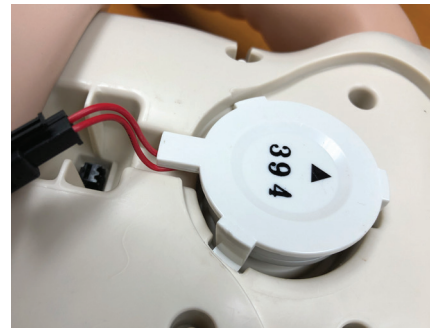


Figure 3

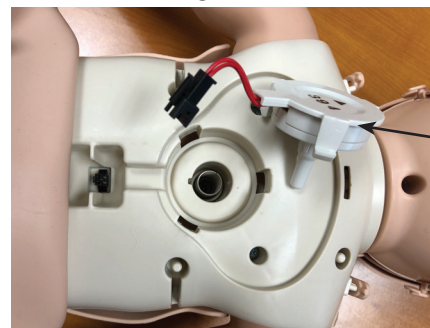

Clicker out
of Manikin

Figure 4